WORKSAFE

WORKWEAR AND PERSONAL PROTECTION

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GB

EN455-1:2001 Medical gloves for single use - Part 1: Requirement and testing for freedom from holes. AQL= Acceptable quality level that indicates the percentage of leaky gloves per production run (or quantity). EN455-2:2009+A1:2011 Medical gloves for single use - Part 2: Requirements and testing for physical properties. EN455-3:2006: Medical glove for single use - Part 3: Requirements and testing for biological evaluation. EN455-4:2009: Medical glove for single use - Part 4: Requirements and testing for shelf life determination.

The glove is in compliance with the EC1935/2004 and 10/2011. *These gloves are approved for use with foodstuffs. Food approval does not guarantee safe handling of all types of foods as there may be a risk of migration between the material and the food in certain cases. Please read Procurator's EC declaration of conformity for more detailed information.

The glove is graded under category I, for minimal risks and comply with the regulations of Directive 89/686/EEC on personal protective equipment. The glove is in compliance with EN 420:2003+A1:2009 General requirements for gloves. The glove is intended for handling foodstuffs and cleaning, and to protect hands from dirt, slight risk and/or protect products/persons from the user. Avoid contact with fatty foods! Before using check that the glove suits the work task intended. Keep unused gloves in the original packaging. To be stored in a cool dark room. With proper storage the gloves have a shelf-life with unchanged product attributes according to the expiry date on the package. New gloves should be checked carefully to make sure that they are damage-free before being used. If there are signs of damage to the product, discard it immediately. Disposable glove must be discarded after use. Used gloves should be disposed of in accordance with the regulations of national or regional controlled industrial waste deposits. The glove material is not known to cause allergic reactions. If irritation

For further information please contact:

 Procurator AB, P.O. Box 9504, SE-200 39 Malmö, Sweden

SE +46 (0)10 60 40 000
DK +45 761 150 00

NO +47 481 122 22
FI +358 (0) 207 768 501

www.procurator.net